m-doc™ Nasal Plug Instructions for Use

Expandable Nasal Sponge with m-doc™

Characteristics

Expandable PVAc (polyvinylacetal) sponge for nose bleeds. Nasal plugs absorb the blood and expand rapidly into a soft sponge. The nasal plugs are impregnated with m-doc™ (micro-dispersed oxidized cellulose) powder to stop bleeding fast. Nasal plugs do not stick to the walls of the nose, allowing easy removal and reducing the risk of renewed bleeding.

MD Medical device.

Indications

Expandable dressing for nosebleeds

Contraindications

- Do not use on patients with a known sensitivity for one of the product's components (PVAc or oxidized cellulose or its derivatives).
- Not recommended to use for children under 5 years

Directions for use

- 1) Remove the nasal plug from packaging and insert into nose. When inserting, ensure some of the product remains outside the nose to allow easy removal. The nasal plug will expand and soften like a sponge as it absorbs blood.
- 2) After a few minutes, when bleeding has stopped, carefully remove the nasal plug from the nose. Dispose in household waste, do not flush.

Note: For children over 5 years, prior to use, the plug can easily be cut to size using scissors.

Precautions

- Do not leave the plug in the nose for more than one hour.
- In the event of prolonged bleeding, seek medical advice.
- Only use one nasal plug into one nostril at a time.
- Do not re-use. Re-use could cause infection.
- Store in a dry location. Protect from sunlight or excessive heat.

If a serious incident occurs involving the device, inform HemCon Medical Technologies CZ, your local distributor, and your local healthcare authority.

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