

HemCon® Dental Dressing PRO

Oral Surgery

Periodontist

General Practice

Oral and Maxillofacial Trauma


Emergency Medicine

Positive Outcomes. STAT.

DENTAL DRESSING FOR TEMPORARY PROTECTION OF ORAL MUCOSAL TISSUE WOUNDS

HemCon Dental Dressing is an ideal practice management tool, allowing for quick patient turns and wound protection following oral procedures.

- **Cost effective:** Ideally sized dressings; helps reduce chair time to maximize patient turns.
- **Less hands-on time:** Adhesive properties help secure dressing to mucosal tissue; primary closure not necessary; may reduce overall need for sutures.
- **Dependable:** Works quickly; offers pain relief; works without the use of thrombin.
- **Easy to Use:** Intuitive application; typically dissolves within 48 hours, no removal necessary.
- **Safe:** Provides a physical barrier for temporary protection of the wound surface; no known contra-indications.

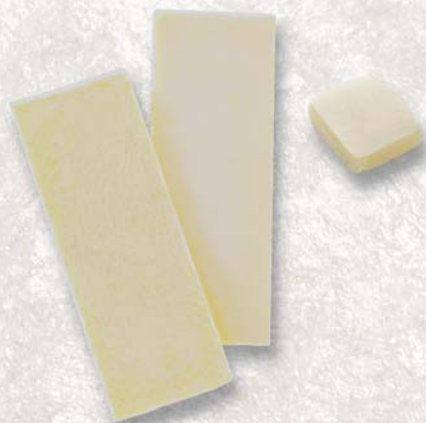

Extraction site on alveolar ridge.



Wound site after three weeks.



 HemCon
by **Tricol** BIOMEDICAL INC

WE PUT THE STAT IN HEMOSTAT

HemCon® Dental Dressing PRO

INDICATION FOR USE

The HemCon Dental Dressing is an oral wound dressing intended as a physical barrier for temporary protection of oral mucosal tissue and to provide pain relief.

HOW HEMCON BANDAGES WORK

The HemCon Dental Dressings become mildly adhesive when in contact with the oral environment, the adhesive properties of chitosan help secure dressing to mucosal tissue, providing a physical barrier for temporary protection of the wound surface while offering pain relief.

APPLICATION GUIDE: 10MM X 12MM

1. If required, trim HemCon Dental Dressing 10mm x 12mm to loosely fit extraction socket. Dressing is highly effective; less is better to minimize potential pain.
2. The dressing requires blood or moisture to adhere to tissue. Both sides of the dressing are active.
3. Using dry instruments, loosely place the dressing into the extraction site (do not pack site or force the dressing into the apex of the socket). The top of the dressing should be flush with the crestal gingiva.
4. Apply gentle pressure with an instrument or place gauze over the dressing and have the patient apply gentle biting pressure for one to two minutes until the dressing adheres.
5. Before discharging patient, visually inspect the site to confirm correct positioning of the dressing.



APPLICATION GUIDE: 1IN X 3IN

1. The HemCon Dental Dressing 1 in x 3in can be trimmed to fit any oral wound.
2. The dressing requires blood or moisture to adhere to tissue. Both sides of the dressing are active.
3. Using a clean, dry instrument, place the HemCon dressing onto the oral wound site.
4. Apply gentle pressure with an instrument on the dressing or by placing gauze over the dressing and then apply pressure for one to two minutes until the dressing adheres.



REMOVAL INSTRUCTIONS

The HemCon Dental Dressing usually dissolves within 48 hours, but may take up to seven days. At seven days the wound site can be irrigated to ensure removal of any residual material.

CAUTION

Never suture HemCon Dental Dressing in place using primary closure. An open wound is required when using this device.

ORDER INFORMATION

Part Number	Part Number	Configuration
HemCon Dental Dressing, 10mm x 12mm	1050	12/bx or 144/cs
HemCon Dental Dressing, 1 in x 3in	1052	12/bx or 144/cs

510K 060363
Tax ID: 81-2091181

MMF-232 Rev. 3 (US) 10/16

CONTACT US DIRECTLY AT

U.S. & Canada Toll Free: 877.247.0196

Phone: +1.503.245.0459

www.tricolbiomedical.com • info@tricolbiomedical.com

**Tricol**
BIOMEDICAL INC

720 SW Washington Street, Suite 200 • Portland, OR 97205-3504