

INTERNAL USE ONLY
— DO NOT PRINT THIS PAGE —

RM7372 Rev 1 - Package Insert, IFU, ChitoGauzeXR PRO, Hospital
MS-RM4018

Size: 8.5" x 5.5" folded to 4.25" x 5.5"

Color: Black

Chito**Gauze**[®]**XR**PRO

HEMCON® CHITOGAUZE®XR PRO | X-RAY DETECTABLE

Intended Use/Indications:

HemCon ChitoGauzeXR PRO is a hemostatic dressing for the external, temporary control of severely bleeding wounds.

ChitoGauzeXR PRO Instructions for Use:

1. Open foil pouch and remove ChitoGauzeXR PRO.
2. Identify and cover the source of bleeding with the dressing, apply direct digital pressure to the source of bleeding.
3. Pack dressing completely into wound track. Use enough ChitoGauzeXR PRO to fill the wound and contact all bleeding surfaces. More than one dressing may be required.
4. If cutting the dressing to a smaller desired size, ensure the radiopaque element is included in the parts you are using.
5. Apply pressure until bleeding is controlled.
6. Wrap and secure dressing on wound to maintain pressure.

FOR USE BY TRAINED EMERGENCY RESPONDERS

Cautions/Warnings:

FOR EXTERNAL USE ONLY. DO NOT IMPLANT.

Do not use if seal is broken and/or contents are wet. Do not apply over eyes. Do not use near the spine. Use promptly after opening. Contains chitosan from shellfish. Single use only. Discard any unused portion of the product. Do not resterilize. Not made with natural rubber latex.

Seek Medical Attention:

- If bleeding does not stop or redness or swelling occurs.
- If you suspect an allergic reaction.

EXPLANATION OF USED SYMBOLS



Manufacturer



Catalogue number



Batch code



Use-by date



Warning



**Do not use if package
is damaged**



Keep dry



Single use only



Sterilized using irradiation



**Single sterile barrier system.
Sterilized using irradiation**



Do not resterilize




Not made with natural rubber latex



Temperature limit

R_x ONLY

Prescription Only

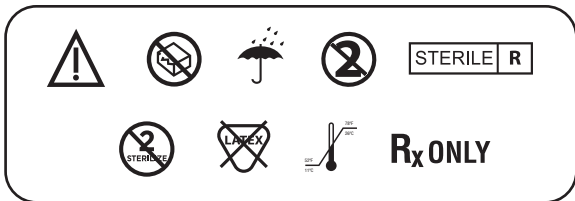

Tricol Biomedical, Inc.

720 SW Washington Street, Suite 200

Portland, OR 97205 U.S.A.

www.tricolbiomedical.com

info@tricolbiomedical.com

+1.503.245.0459

Toll Free (U.S. & Canada) 877.247.0196


Tricol

BIOMEDICAL INC

RM7372 Rev 1 02/2024