**INTERNAL USE ONLY**  
**— DO NOT PRINT THIS PAGE —**

HemCon

GuardaCare®

PRO

 Manufactured for:  
Tricol Biomedical, Inc.  
720 SW Washington Street, Suite 200, Portland, OR 97205 U.S.A.  
+1.503.245.0459 | Toll Free (U.S. & Canada) 877.247.0196  
www.tricolbiomedical.com | info@tricolbiomedical.com  
Your feedback is important to us.  
Please visit www.tricolbiomedical.com/feedback.

  :CEpartner4U, Esdoornlaan 13,  
3951DB Maarn, The Netherlands  
www.cepartner4u.eu

FOR USE BY TRAINED EMERGENCY RESPONDERS

RM702 Rev. 6 05/25

ENGLISH

Hemcon® GuardaCare® PRO

**Intended Use / Indications:**  
HemCon GuardaCare PRO is a hemostatic dressing for the external, temporary control of severely bleeding wounds.

**GuardaCare PRO Instructions for use:**

1. Open foil pouch and remove GuardaCare PRO.
2. Place or pack dressing completely over the source of bleeding. Make sure to contact all bleeding surfaces. More than one dressing may be required.
3. Apply pressure until bleeding is controlled.
4. After bleeding has stopped, secure GuardaCare PRO with appropriate dressing (not included).

FOR USE BY TRAINED EMERGENCY RESPONDERS

**Cautions / Warnings:**  
**FOR EXTERNAL USE ONLY, DO NOT IMPLANT.** Do not use near the spine. Do not use if seal is broken and/or contents are wet. Do not apply over eyes. Use promptly after opening. Contains chitosan from shellfish. Single use only. Discard any unused portion of the product. Do not resterilize. Not made with natural rubber latex.

- Seek Medical Attention:**
- If bleeding does not stop or redness or swelling occurs.
  - If you suspect an allergic reaction.

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